

DR WOUND BIOHEAL GEL

20ml

FIRST AID

MDA NO. : GC7343919-29139
Registered under Act 737



Bioheal Gel is a primary biological dressing consist of natural non-animal origin biopolymers. It maintains a moist environment and accelerates the healing rate.

*It attracts necessary cells and proteins to generate faster healing process and forms secondary skin to protect the wound while reducing scars and keloids formation.¹

Active Ingredient : Chitosan Biopolymer

***Feature :**²

Antibacterial

Reduces scarring and prevents keloid formation

Promotes moist environment

Forms secondary skin to protect the wound

Fast wound healing and closure

Non-toxic

Indication

- Minor Cuts and Scratches
- First and second degree burns
- Abrasion
- Post-Surgery Wound
- Episiotomy
- Cesarean Wound
- Circumcision
- Diabetic Ulcer
- First Aid

Bioheal Gel ialah bahan biologi lapisan pertama yang terdiri daripada biopolimer semula jadi bukan haiwan. Ia mengekalkan kelembapan di tempat luka dan mempercepat kadar penyembuhan.

Ia menarik sel dan protein yang diperlukan untuk menggalakkan proses penyembuhan yang lebih cepat dan membentuk lapisan kulit kedua untuk melindungi luka di samping mengurangkan parut dan pembentukan keloid.

Ramuan Aktif : Biopolimer Chitosan

Kelebihan :

Antibakteria

Mengurangkan parut dan mencegah pembentukan keloid

Mengekalkan kelembapan di tempat luka

Membentuk lapisan kulit kedua untuk melindungi luka

Menyembuh dan menutup luka dengan cepat

Tidak toksik

Indikasi

- Luka kecil dan calar
- Luka terbakar darjah pertama dan kedua
- Lelasan
- Luka selepas pembedahan
- Episiotomy
- Luka pembedahan Cesarean
- Berkhatan
- Ulser diabetes
- Pertolongan Cemas

Instruction For Use / Cara Penggunaan :



1. Clean the wound surface.
Bersihkan luka tersebut.
2. Apply a layer of Dr Wound Bioheal Gel First Aid.
Sapukan Dr Wound Bioheal Gel First Aid ke atas luka.
3. Cover the area with secondary dressing if needed.
For minor cuts, there is no need for secondary dressings.
Balut dengan pembalut jika perlu. Untuk luka kecil, tidak perlu dibalut.

*1. Based on clinical study.

*2. Based on lab testing.

FINISHED PRODUCT AND COMPANY IDENTIFICATION

Product Name : Dr Wound Bioheal Gel First Aid		Product intended use : Promotes healing for stiches, cuts and burn	
Product Size : 20mL			
Manufactured & Distributed by : Farmasia Sdn. Bhd. (772557-D) B2-2 Ribosom Block, UKM-MTDC Technology Centre, 43650 Bandar Baru Bangi, Selangor, Malaysia.		Central Office : B2-2, Ribosom Block, UKM MTDC Technology Centre,	
City : Selangor	Province : Bandar Baru Bangi	Postal Code : 43650	Emergency Phone : +603 8925 1888
Fax : +603 8925 2888	Website : www.farmasia.com.my	Email Address : enquiries@farmasia.com.my	
Date PSDS Prepared 11/06/2017	PSDS Prepared By : Research & Development Department		

COMPOSITION / INFORMATION ON INGREDIENTS

Active Ingredient : Chitosan Biopolymer		
Name	Catalog Code	Volume (mL)
Dr Wound Bioheal Gel First Aid	FFP001.020FA	20
Toxicological Data of Ingredients : This product is non-toxic		

FIRST AID MEASURES

Skin Contact : This product is intended to be in contact with skin when used as directed. If irritation develops, seek medical attention.	
Eye Contact : The eyes should be washed immediately with plenty of water. If irritation continues , consult a physician.	
Ingestion : Do not ingest. If ingested, contact a physician.	Inhalation : Not applicable.

HANDLING AND STORAGE

Handling procedures and equipment	Handling procedures and equipment : Protect package against physical damage. Keep out of children's reach.
Storage requirements	Store at (5-30)°c . Keep away from moisture and sunlight.

DISPOSAL CONSIDERATIONS

Waste treatment methods : There is no special waste treatment method.
Product/ Packaging disposal : Disposal of product and its packaging require no special method.